# HEI-KA-KOMP®

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# HEI-KA-KOMP®

## Oncological cooling glove

The company Reus Medizinische Produkte is the manufacturer of the high-quality HEI-KA-KOMP® multiple cold warm compresses, which have proven themselves in the hospital sector due to their long service life.

That is why we have included the HEI-KA-KOMP® cooling glove in our product range. The special was particularly developed for the application of the cooling glove. It maintains its firm consistency over the entire temperature spectrum and thus guarantees an even temperature distribution with a long cooling effect.

By using the HEI-KA-KOMP® cooling glove, you avoid nail damage and skin irritations during the chemo treatment.

During chemotherapy, about 50% of all patients suffer from skin changes and damage to fingernails and toenails which can lead to pain. Edema formation or even loss of the nail can occur. In order to prevent these consequences and the premature termination of chemotherapy, we recommend wearing HEI-KA-KOMP® cooling gloves during treatment as it will significantly reduce these symptoms on the hands and feet.

HEI-KA-KOMP® cooling gloves offer the following advantages to the patient:

1. Very cost-effective compared to conventional gel gloves
2. 100% hygiene due to individually disinfectable gel pads
3. Uniformly high cooling capacity due to special gel
4. Simple and uncomplicated application
5. Each part can be reordered separately
6. Easy to fix by means of flex tape with Velcro fastener
7. High wearing comfort for the patient and easy handling for the nursing staff
8. Disposal in household waste possible due to atoxic substances

### Application

Store the gel pads in the freezer and simply place them in the two cooling pockets of the fleece gloves for use. Make sure that the gel pad slides all the way to the front. The gel pad remains elastic down to -20°C.

The hands and feet must not hurt during application!  
Attention: Risk of frostbite!

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